

## 10.4 Cleaning



**Caution**

Make sure to fix the head securely before cleaning.  
It may result in injury if the head is fallen.

- **Cleaning of the measuring window**

If the measuring windows or VD check windows have stains, wipe lightly with the soft cloth. In this case, handle it carefully and do not scratch it.

- **Cleaning of the cover**

If the cover has stains, wipe it with a dry cloth.

If it has heavy stains, wipe it by using a little amount of water or neutral detergent



Do not use organic solvent such as thinner  
because it damages the surface of the device.

- **Cleaning of the LCD touch panel**

Use the absorbent cotton or soft cloth when the LCD touch panel has stains.


- **Cleaning of the headrest**

1) Remove the headrest (refer to the figure on the right).

2) Wipe it with the clean cloth.

If it has heavy stains, wipe it with a diluted neutral detergent solution. For disinfecting it where the examinee may contact, use the ethanol for disinfection.

※ Ethanol for disinfection contains 76.9 to 81.4vol% of ethanol ( $C_2H_6O$ : 46.07) at 15°C (specific gravity).

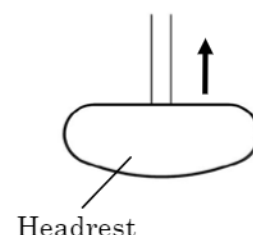

3) Put the headrest back together.

- **Cleaning of the face shields**

1) Remove the face shields.

It can be easily detached since they are fixed by magnets.

2) Wipe it with the clean cloth.

If it has heavy stains, wipe it with a diluted neutral detergent solution. For disinfecting it where the examinee may contact, use the ethanol for disinfection.

※ Ethanol for disinfection contains 76.9 to 81.4vol% of ethanol ( $C_2H_6O$ : 46.07) at 15°C (specific gravity).

3) Put the face shield back together.

Basically, it is not necessary to replace the headrest and the face shields. They comply with ISO 10993-1.