

8.5.3 Replacing Fuse

- ① Turn off and raise the Non-Contact Tonometer HNT-7000 with two arm carefully.
- ② Remove the Power cord
- ③ Pick the fuse holder out from the Power inlet
- ④ Exchange the fuses
- ⑤ Insert the fuse folder

8.6. Cleaning Equipment

- ① The Equipment should be kept as clean basically. Do not use the solvents such as strongly volatile substance, thinner, benzene, etc.
- ② Put some soapy water to the soft cloth, and twist the water out of the colth. Then, polish each part of the equipment. (RcooNA , NaOH, potash..)
- ③ As polishing the parts of lens or glass, get rid of dusts on the surface of lens with air tube and use a dry cloth.

8.7. As changing the installation place of the equipment

- ① Off the power switch of main body.
- ② Take the power connection cable apart.
- ③ Move it while maintain the horizontality of it by holding the bottom of the main body.