


IVS Electronic measurements tools : Eyeruler 2

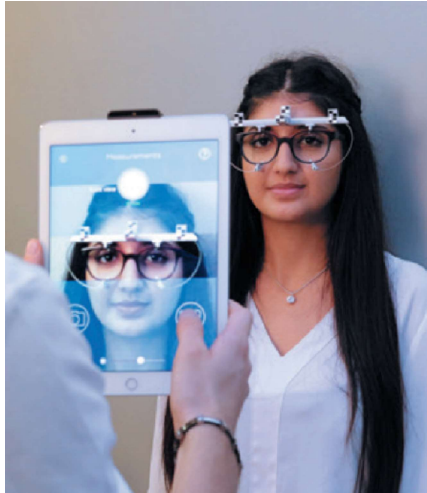

- Measurements: all necessary and personalised parameters are taken with a single solution
- Distance ECP /patient $\geq 1\text{m}$
- The patient does **not need to touch/hold a device** for far vision measurements
- The ECP has **no physical contact** with the patient whilst taking measurements
- The patient stands at 75 cm from the device



Manual measures



- Distance ECP/patient = **35 cm max**
- Device **must be in contact with the patient's face** (nose and forehead)
- Patient needs to hold the device



- Distance ECP/patient = arm's length
- ECP **risks touching the patient's face** whilst holding the ruler
- ECP needs to be close to the patient to read the ruler's markings



Frame with clip handling:

When adjusting the frame (previously cleaned) and positioning the clip on the frame, the ECP must wear disposable gloves and as well as a protection mask.

Cleaning recommendations:

List of items to be cleaned before and after each set of measurements: the frame, the clip, the iPad® *.
To clean all electronic devices we recommend using hydroalcoholic wipes, the clip can be cleaned with hydroalcoholic wipes or a soft, dry cloth with alcoholic solution.

Warning:

When cleaning all electronic devices do not let any liquid enter electronic parts, please use hydroalcoholic wipes.